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# MEDICALTEST

## DIAGNOSTIC TEST SPECIALIST

## ENGLISH PREGNANCY TEST – INSTRUCTIONS

Pregnancy test strips are samplers which contain a reactive for determining the presence of hCG (Human Chorionic Gonadotropin) in urine samples. This kit is used to obtain a visual, qualitative result for early pregnancy detection. Human Chorionic Gonadotropin (hCG) is a hormone that is secreted by the developing placenta shortly after fecundation. Appearance of hCG after conception and its subsequent rise in concentration during early gestational growth make it an excellent marker for early detection of pregnancy.

### PROCEDURE

To begin the test, open the sealed aluminium bag by tearing it along the groove. Take the test out of the bag. Note: Morning urine generally contains a higher concentration of hCG, and is therefore the best sample when taking the urine test. Nonetheless, urine samples collected during the rest of the day can be utilised.

Collect a sample of your urine in a clean, dry container. Take the strip vertically, holding it by the coloured part.

Immerse the strip in the urine sample, with the tip of the arrow pointing towards the urine. Do not submerge the sampler beyond the line. Keep the strip submerged in the urine for 3 seconds. You will notice the strip getting a dark tint, and this shade climbs up the strip. Immediately afterwards, withdraw the strip from the urine and place it on a clean, dry, non-absorbent surface (do not place it on a napkin, toilet paper or the like).

Note: In the off chance that the tint of the test does not reach the results area, immerse the tip of the reactive strip back in the urine as previously indicated until the dye starts moving towards the results area.

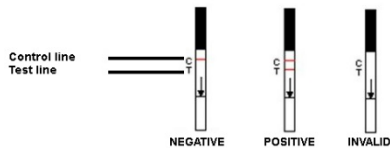
Wait for the coloured lines to appear. Depending on the hCG concentration in the test sample, positive results can be seen within just 40 seconds. However, in order to confirm negative results, it is necessary to wait the full reaction time, which is 5 minutes. Do not read the results after the 5 minutes specified have passed.

### RESULTS

**Negative:** only one coloured line appears in the control region. No line appears in the test region. This indicates pregnancy has not been detected.

**Positive:** one coloured line appears in the control region, and another line in the test region. Presence of both lines indicates you are pregnant. Important: the intensity of the colour of the test lines may vary. Depending on the different stages of pregnancy, there are different concentrations of the HCG hormone. It is quite common for the line in the test region to be lighter than the line in the control region. Any line or clustering of colour or contrast that appears at the joint of both lines must not be confused with the test line.

**Invalid:** no line is visible. The control line does not appear if enough volume of the sample has not been taken, or if proper procedures were not followed when performing the test. Repeat the procedure with a new pregnancy test. Please, read the instructions for performing the test and follow them carefully.

### WARNINGS AND USAGE LIMITATIONS

Pregnancy tests should be stored at ambient temperature (18 to 30 °) in the sealed aluminium bag. They should be kept away from the sun, moisture and heat. Do not use the pregnancy test after its expiration date.

Just as with any diagnostic test, a definitive clinical diagnosis should not be based on the results of a single test. If in doubt, consult your physician.

Negative results may appear if a test is performed at very early stages of the pregnancy. This may be due to an extremely low level of hCG concentration. In these cases, testing should be repeated with a new sample obtained two days later.

If a urine sample is too diluted (due to having too high a concentration of water), this may cause low levels of hCG in the urine. If the result is negative under these circumstances and you still suspect you may be pregnant, try obtaining a sample of the first morning urine and repeat the test.

Instructions for the test must be followed carefully to ensure precise results. Please, read the instructions and follow them to obtain proper results.

HCG urine pregnancy tests are for external use only.

Please, consult your physician to verify the results of the diagnostic or if you receive unexpected or inconsistent results

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